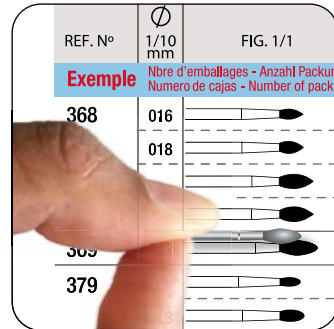
INSTRUMENTS ROTATIFS DENTAIRES

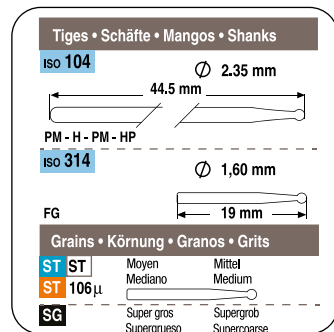
DENTAL ROTIERENDE INSTRUMENTE - STRUMENTI ROTATIVI DENTALI - DENTAL ROTARY INSTRUMENTS

1

Sélectionner formes et diamètres (Echelle 1:1)
Form und Durchmesser auswählen (Maßstab 1:1)
Selezionare forme e diametri (Scala 1:1)
Select forms and diameters (Scale 1:1)

2

Choisir tige et grain (diamants)
Schaft und Körnung (Diamanten) wählen
Selezionare gambo e grano (diamanti)
Choose shank and grit (diamonds)

3

FIG. 1/1	P	314	314	314	314	314	204
		ST	SG	G	F	C	ST
1	1						
1	1						
3	3						
3	3						
3	3	6		3			
1	1						
1	1						

Inscrire la quantité d'emballages
Verpackungsmenge eintragen
Indicare la quantità di imballaggi
Write the quantity of packages

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Hygiene recommendations, disinfection, cleaning, sterilization and storage.

1. Instruments must always be disinfected, cleaned and sterilized before they are used on patients.
2. Soak all used instruments in a disinfecting solution which is combined if possible with a proteolytic enzyme. Follow instructions and observe concentrations and immersion times given by the solution manufacturer. Do not use too aggressive products (ex. hydrochloric acid...) which may corrode the devices.
3. Place the devices such as to avoid any contact between them. Immerse in an ultrasonic bath, with a detergent solution having cleaning properties. Follow instructions and observe concentrations and time given by the manufacturer.
4. After abundant rinsing with clean, demineralised or distilled water, immediately dry the instruments to avoid any corrosion.
5. Place the instruments in stands and trays suitable for sterilization and sterilize at 134°C for 18 min.
6. After sterilization, keep the devices in a sterile packaging, in a dust-free place.

Axis Dental Sàrl

Chemin du Cloalet 4 - 1023 Crissier - Switzerland

Tél. +41 (0)21 633 06 33 - Fax + 41 (0)21 633 06 39 - mail@axis-dental.ch - www.axis-dental.ch

Symboles utilisés sur l'étiquetage:
Erklärung der Symbole auf der Verpackung:
Simboli utilizzati nell'etichettatura:
Symbols used on labelling: