

1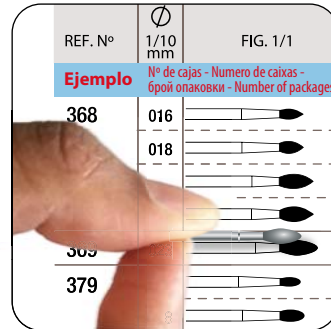

Seleccione las formas y diámetros (Escala 1:1)  
Selecione as formas e diâmetros (Escala 1:1)  
Изберете форми и диаметри (Мащаб 1:1)  
Select forms and diameters (Scale 1:1)

2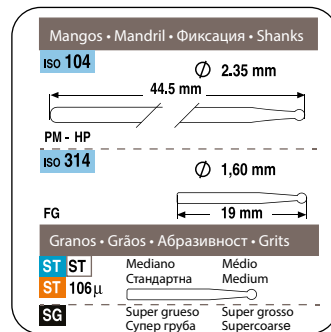

Seleccione el mango y grano (fresas diamantadas)  
Selecione o mandril e o grão (brocas diamantadas)  
Изберете цанга (вид наконечник) и абразивност (за диамантени)  
Choose shank and grit (diamond burs)

3

FIG. 1/1	P	314 ST	314 SG	314 G	314 F	314 C	204 ST
caixas - Number of packages							
	1						
	1						
	3						
	3						
	3	6	3				
	1						
	1						

Indique la cantidad de cajas  
Indique a quantidade de caixas  
Напишете количеството на опаковките  
Write the quantity of packages

No olvide escribir su dirección / Não se esqueça de escrever a sua morada  
Не забравяйте да регистрирате вашия адрес / Don't forget to write down your address

# Fresas de Carburo de Tungsteno / Brocas em Carboneto de Tungsténio 1

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## Hygiene recommendations, disinfection, cleaning, sterilization and storage.

- Instruments must always be disinfected, cleaned and sterilized before they are used on patients.
- Soak all used instruments in a disinfecting solution which is combined if possible with a proteolytic enzyme. Follow instructions and observe concentrations and immersion times given by the solution manufacturer. Do not use too aggressive products (ex. hydrochloric acid...) which may corrode the devices.
- Place the devices such as to avoid any contact between them. Immerse in an ultrasonic bath, with a detergent solution having cleaning properties. Follow instructions and observe concentrations and time given by the manufacturer.
- After abundant rinsing with clean, demineralised or distilled water, immediately dry the instruments to avoid any corrosion.
- Place the instruments in stands and trays suitable for sterilization and sterilize at 134°C for 18 min.
- After sterilization, keep the devices in a sterile packaging, in a dust-free place.

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